

RELEAF INSTRUCTION MANUAL

SETUP & PATIENT USE

- 1) Fold the two Leaf halves together by sliding the protruding dovetail clip into the opposing groove at the leaf tip. Squeeze the two halves of the leaf together slightly to ensure the middle lock tab is inserted into the hole in the center of the leaf. (See left image below). Attach the Leaf to the “U” end of the Hose with the lower suction hole facing away from the hose. Ensure the Leaf is securely attached.
- 2) Attach the free end of the Quick-Disconnect Hose to the Quick-Disconnect Adapter of the HVE Attachment Hose (See right image below). Insert the coupler at the other end of the HVE Attachment securely into an open HVE port. It may be necessary to wet the O-ring inside the unused port for the adapter to slide in easily. It also may be necessary to replace the O-ring if you find the coupler coming loose.
- 3) Place ReLeaf into the patient’s oral cavity with the Leaf positioned between the teeth and cheek.



When transitioning between patients, remove the Quick-Disconnect Hose, wipe down the short 8 inch hose with a disinfectant wipe, reattach the 8in hose to the adapter, and attach a new Leaf.

FOR FURTHER INSTRUCTION ON HOW TO SET UP RELEAF FOR PATIENT USE, PLEASE REFER TO THE INSTRUCTIONAL VIDEOS INCLUDED ON OUR WEBSITE AT www.ReLeafDental.com.

STERILIZATION, DISPOSAL & STORAGE

Disposable Leaf

Disposable Leaves are meant for single use only. Due to the nature of the material and design of the Leaves, they are unable to be sterilized and reused.

Reusable Quick-Disconnect Hose (Autoclavable)

Take the Quick-Disconnect hose and place it in the ultrasonic for 15 minutes, rinse with water, insert the hose into a large autoclave pouch, then put the hose through a wrapped cycle in the autoclave.

Reusable HVE Attachment Hose with Quick-Disconnect Adapter (Autoclavable)

Run evacuation cleaner through the HVE Attachment Hose per your normal office cleaning schedule. Place the hose in the ultrasonic for 15 minutes, rinse with water, insert the hose into a large autoclave pouch, then put through a wrapped cycle in the autoclave.

We recommend wiping the hose down and running the short hose through a wrapped autoclave cycle between patients.

We recommend running evacuation cleaner through the hose at least once per week. It is not necessary to run evacuation cleaner through the hose between each patient.



Storage

Please ensure ReLeaf is stored at room temperature. The Quick Disconnect is specifically designed to be placed in an available rack slot when not in use. (See Right)



TROUBLE SHOOTING

Proper ReLeaf Position

By placing the leaf in the location shown here in the picture, the leaf will be in the prime location to deliver a consistent and comfortable suction during the dental procedure. PLEASE ENSURE that you place ReLeaf on the same side you are working and ensure that the patient tilts their head toward the side you are working on for optimal performance.

To assist in evacuation when a small amount of water and saliva to build up, you can have the patient slowly close down when needed (we recommend teeth before lips). When the patient closes their jaw or occludes their teeth together, it pushes the tongue away from ReLeaf. Fluid is then forced to the cheeks where ReLeaf can evacuate the remaining fluid.



Large Oral Cavity

When placing ReLeaf into a large oral cavity, please direct patient to close down slightly to ensure that the sides of ReLeaf fit comfortably into the vestibule area. Some patients can open wider than ReLeaf's designed maximum width. This causes ReLeaf to dislodge out of place towards the midline. The solution is to inform patients with a large max opening to relax their jaw until the sides of ReLeaf begin to enter the buccal area. ReLeaf is designed to allow as much working space as possible. The designed width of ReLeaf, with the combination of a large max opening, still allows the practitioner plenty of room to perform the necessary treatments.

CONTACT & REORDER INFORMATION

To order ReLeaf refill packs:

www.ReLeafDental.com
(360) 258-1971

Hours of operation:

Monday-Friday, 9am-5pm

If you have questions or feedback, please e-mail us at: info@innovatingdentistry.com

Stay connected!

From updates on upcoming enhancements to sneak peeks of upcoming new products, simply e-mail us at info@innovatingdentistry.com to be added to our e-mail list. You will also receive discount coupons!

Follow us:

ReLeafDental.com
InnovatingDentistry.com
Facebook.com/InnovativeDentalTech
Instagram.com/InnovativeDentalTech

Thank you for choosing ReLeaf!

